


Give your wound management a better choice

Bodyguard™

Singe Negative Pressure Wound Therapy Kit

Biocompatible Polymer Semipermeable Membrane

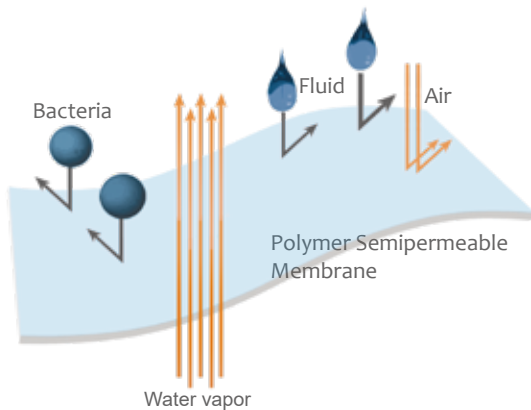

- Better isolation from bacteria.
- Hypoallergenic and non-irritant.
- More air permeability and more comfortable.


More Effective Sucking Disc

Structure design based on fluid dynamics, reducing block risk.

Ensure even distribution of negative pressure to every outer cavity, which can be monitored real-time.



Biocompatible Reticulated Foam

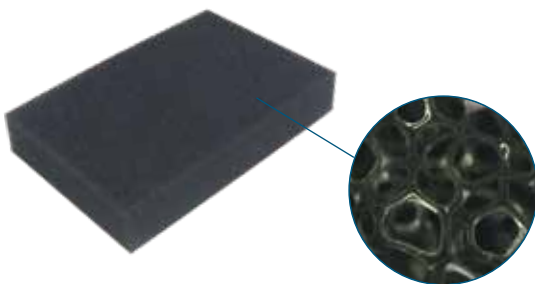

Molecules in lower polarity, preventing diffusion of bacteria with polarity.

Inhibition of inflammation, improving the efficiency of wound healing.

Uniform pore size of foam.

Complete contact with wound surface for even pressure distribution.


More stable structure with thermosetting property.

Easy to peel off, hard to be broken, lower rate of residue.


Indications

Trauma & Orthopedics	Burn Surgery	General Surgery & Cardiothoracic Surgery	Hand and Foot & Microsurgery
<ul style="list-style-type: none"> • Open fractures • All kinds of defect and damages of skin or soft tissues • Avulsed wound or egloving injury • Osteofascial compartment syndrome • Chronic osteomyelitis and wound caused by amputation 	<ul style="list-style-type: none"> • Burn & scald, hot crush injury and high-explosive injury etc. • Extensive burns • Deep second degree burn after eschar shaving and third degree burn after eschar excision (after shock period with vital signs stable) • Wound surface before transfer of grafts 	<ul style="list-style-type: none"> • Postoperation of mastocarcinoma radical operation • Postoperation of rectal carcinoma radical resection • Unhealing incision or stoma • Chronic empyema, bronchopleural fistula and fistula of esophageal anastomosis etc. 	<ul style="list-style-type: none"> • Wound surface preparation before transfer of flaps • Grafts • Crush syndrome • Chronic wounds, including diabetic foot ulcers


Operation Steps

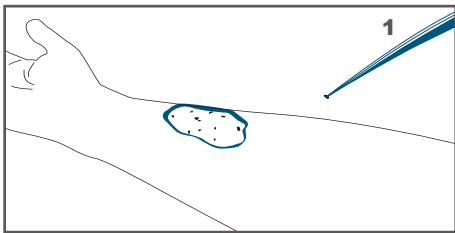


Step 1 Scrub slightly the skin around after debridement.

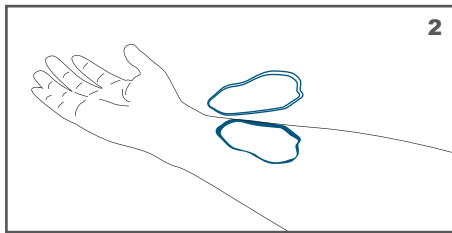


Step 2 Cut and put the foams together and cover them over the wound, make sure the foams fit well.

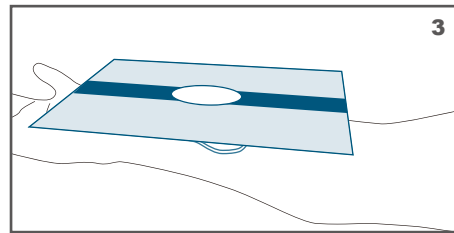

Step 3 Apply the sealing drape.

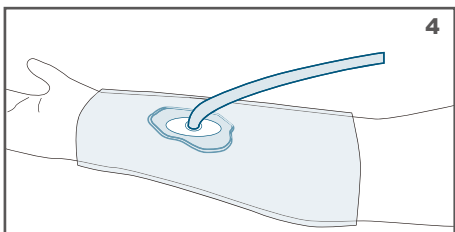

Step 4 Place the sucking disc.

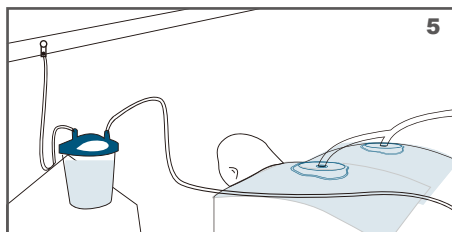

Step 5 Connect with the Wound Continuous Drainage Suction Unit, adjust to appropriate negative pressure.

Optional Step: Perfusion or drugs can be applied if needed, as the operation is easy and can obtain a better healing result.

Ordering Information

Description

Bodyguard II™

7.5×5cm	18×6cm	10×10cm	18×12cm	15×15cm	30×20cm	60×40cm
10×5cm		15×10cm	24×12cm	20×15cm	40×20cm	
15×5cm		20×10cm		25×15cm	50×20cm	

Quantity

1 PC

Drainage Suction Unit

- Size: 200mm*130mm*225mm (±10) Weight: 2kg (±0.5)
- Two operation modes---Continuous Operation and Inter mittent.
- Better design with less noise, the noise is controlled under 50dB(A), which will not disturb the patients at all.
- Long standby, capable of continuous work with less noise.
- There is overflow protection device, which can prevent the exudate sucked from entering into machine, avoiding secondary pollution.



Designed for mobile patients



ZENER MEDTEC (CHANGZHOU) CO., LTD

Bldg 9D, Jinton International Industrial Park No. 8 Xihu
Road Wujin Hi-Tech Industry Zone Changzhou, China

86-0519-83800520 info@zenermed.com

www.zenermed.com